KerraPro™

Pressure Reducing Pads SHEET

10x10cm
(4x4in)

thickness
1.2cm
(0.5in)

silicone

x1

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KerraPro™ Pressure Reducing Pads - Sheet

Description

KerraPro pressure reducing pads are made from flexible, hardwearing silicone and protect against potential skin breakdown in patients with low levels of mobility, or on areas where medical devices may cause damage to the skin due to pressure. Available in a variety of shapes and sizes KerraPro is extremely versatile for protecting at-risk areas.

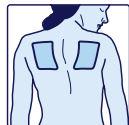
Indications

KerraPro is indicated for the prevention of pressure ulcers in at-risk patients as part of a pressure ulcer prevention programme.

KerraPro Sheet can be used flat or folded and you can even cut the sheets to a required shape or size.

KerraPro Sheet can be used to cushion and protect the head and shoulder blades, reduce pressure between the knees, ankles and under the chin.

KerraPro Sheet can also be used for hand contractures.

Shoulder Blades

Knees & Ankles

Wrists

Contraindications

- Do not use if you have a known sensitivity to silicone. If skin reaction occurs, remove product from skin and seek medical advice.
- KerraPro should not be used on broken skin. It must only be used on healthy, intact or recently healed skin.

Precautions

- It is advised that KerraPro should be cleaned at least once a day. Please ensure you dry KerraPro thoroughly before putting it back in place.
- It is recommended that you wash and dry the skin underneath KerraPro daily to ensure that any moisture build up or contaminants are removed.
- KerraPro is not a wound dressing.
- Skin underneath the KerraPro should be inspected regularly. Only reuse KerraPro while in its original condition. If any damage occurs or the pad loses its tack please replace with a new KerraPro.
- If KerraPro is used on the face, take care not to obstruct nose or mouth airways with the product.

- If the skin condition deteriorates under KerraPro, remove product from skin and seek medical advice.
- Do not keep in place using compression garments.

Method of use

- Remove KerraPro from outer packaging and apply directly to the affected, at-risk area(s). Ensure that the skin is clean and dry prior to application.
- KerraPro should be placed directly on the skin.
- Silicone has a natural 'tack' which helps keep the pad in position. An appropriate retention method may also be needed to keep the pad securely in place. KerraPro can be kept in place by using underwear or socks, a light retention bandage or tape.
- KerraPro can be re-used on the same patient. Wash with soapy water. Ensure dry before re-application.
- When KerraPro is not in use, clean and dry the pad and place back into the outer packaging.
- Dispose in normal waste if used in a household. Dispose in special clinical waste if used in a clinical establishment.

Caring for KerraPro

- It is recommended that KerraPro should be cleaned daily.
- The following methods can be used:
 - Using soap and water - Moisten the pad with soap, massage to remove dirt then rinse and dry thoroughly.
 - Autoclave - can withstand autoclave temperatures of 121°C.
 - Dilute disinfectant - Wipe the surface clean (do not immerse) and rinse to avoid any irritation.

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