

All Purpose Boot (APB)

Product Description

Kerraped[®] is used when a patient has bandages or dressings used to treat the leg, which makes it difficult to wear normal footwear. Kerraped has a breathable upper with a black EVA insole and an abrasion resistant outer sole. Kerraped should only be used on a single patient.

- 

1. Undo Velcro straps and slip your foot in the boot under the 'tongue'
- 

2. Make sure your foot is pushed back against the heel of the boot
- 

3. Place the 'tongue' in position and secure the Velcro straps across the instep of the foot making sure it is secure but comfortable.
- 

4. Secure the Velcro strap around the ankle. Your foot should be securely in the boot and feel comfortable. Using scissors, remove any excess strap.

Warnings & Precautions

Kerraped[®] should only be used when the foot is bandaged. If the shoe becomes soiled, it can be washed with warm soapy water.

Do not machine wash or tumble dry. Do not use if you have a known sensitivity to any of the components of Kerraped. Dispose according to local procedures and guidelines.

Dressing Change & Removal

Your nurse or healthcare professional will advise you how long you should wear Kerraped[®].

Distributed by:
Crawford Healthcare Ltd
King Edward Court
King Edward Road, Knutsford
Cheshire, WA16 0BE, UK
T: +44(0) 1565 654 920
F: +44(0) 1565 654 117


DARCO (Europe) GmbH
Gewerbegebiet 18
D 82399 Raisting
Germany


crawford
HEALTHCARE
www.crawfordhealthcare.com

Email: info@crawfordpharma.com

Kerraped[®] is a trade mark of Crawford Woundcare Ltd.
© Copyright Crawford Healthcare Ltd, 2013.

